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- Bright 6.5“ TFT display with wide viewing angle • resolution of 800 x 480 pixel and LED backlight • interactive operation via touch sensor field and icons • hands-free or handset mode • 2 unit foot positions for different angles of inclination or wall mounting • IP interface to the system switch SWI9 • call indication and call query (incl. speech) according to priority.

Description

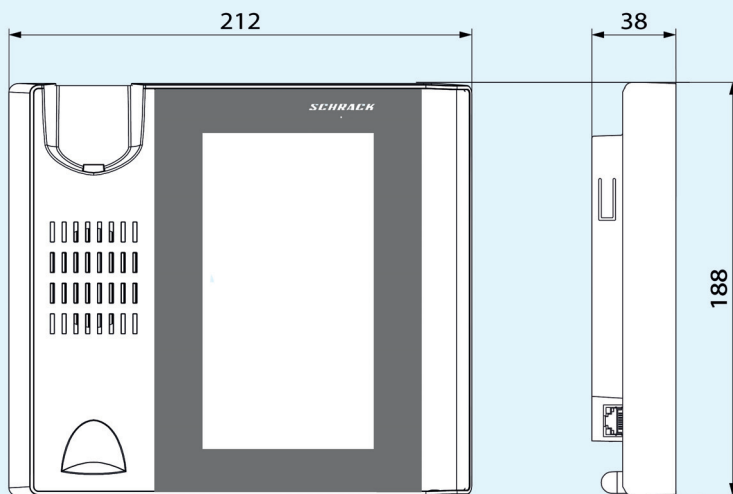
For use as a communications and information centre within a ward and for use in the relevant staff room.

Range of functions:

- Displaying date and time,
- The sum of number of currently outstanding calls, reminders and also possible faults are permanently displayed,
- Indication of all set presences with the staff category and indication of all reminders according to VDE0834,
- Indication of all calls in accordance with VDE0834,
- Indication of events, which have been forwarded from foreign systems into the system,
- All call indications follow automatically the priority for indication, starting with the highest priority:
 - exact type of call e.g. bed call incl. bed number, WC call, doctor call etc.,
 - exact call location with information about the individual room and ward description and the care group that may have been assigned,
 - emergency calls,
- Direct dialling of communications terminals and patient terminals,
- Collective announcements; Staff announcements, separately for all three staff categories,
- Ward interconnections,
- Activation of group-based nursing care and centralised operation.

Staff Terminal ST-TOUCH. Staff terminal ST-TOUCH.

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Technical data

Supply voltage:	24V= (17.5V up to 30V)
Current consumption:	typ: 130mA/24V (3.12W) at low backlight, no call typ: 375mA/24V (9W) at max. backlight max: 500mA/17.5V (8.75W) at max. backlight (max. cable length to the system switch SWI9: 38m)
Connector:	RJ45
LAN interface:	100BASE-TX interface with L3-PoE connection to the SWI9
Reed-contact:	Detects handset launched and switches automatically between handset and speakerphone
Ambient temperature:	0 up to +35°C
Relative humidity:	10 up to 90%
Air pressure level:	≥80 kPa, up to 2000m above sea-level
Protection class:	IP30, VDE0834 environmental Class I
EMC:	EN50081-1 emission for residential use EN50082-2 emission for industrial environments
Product safety:	EN60950
Radio suppression:	EN55022B
Case material:	plastic ABS
Case color:	white RAL 9016
Dimensions:	188 x 212 x 38 mm (LxWxD)

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© Schrack Productions - K-DB-0079DE-EN-ST-TOUCH_V1.1
SCHRACK SECONET AG