




VISOCALL IP. Push Button BT-B.

User Manual.

HEALTH CARE

SCHRACK
S E C O N E T

Patient Handset BT-B.

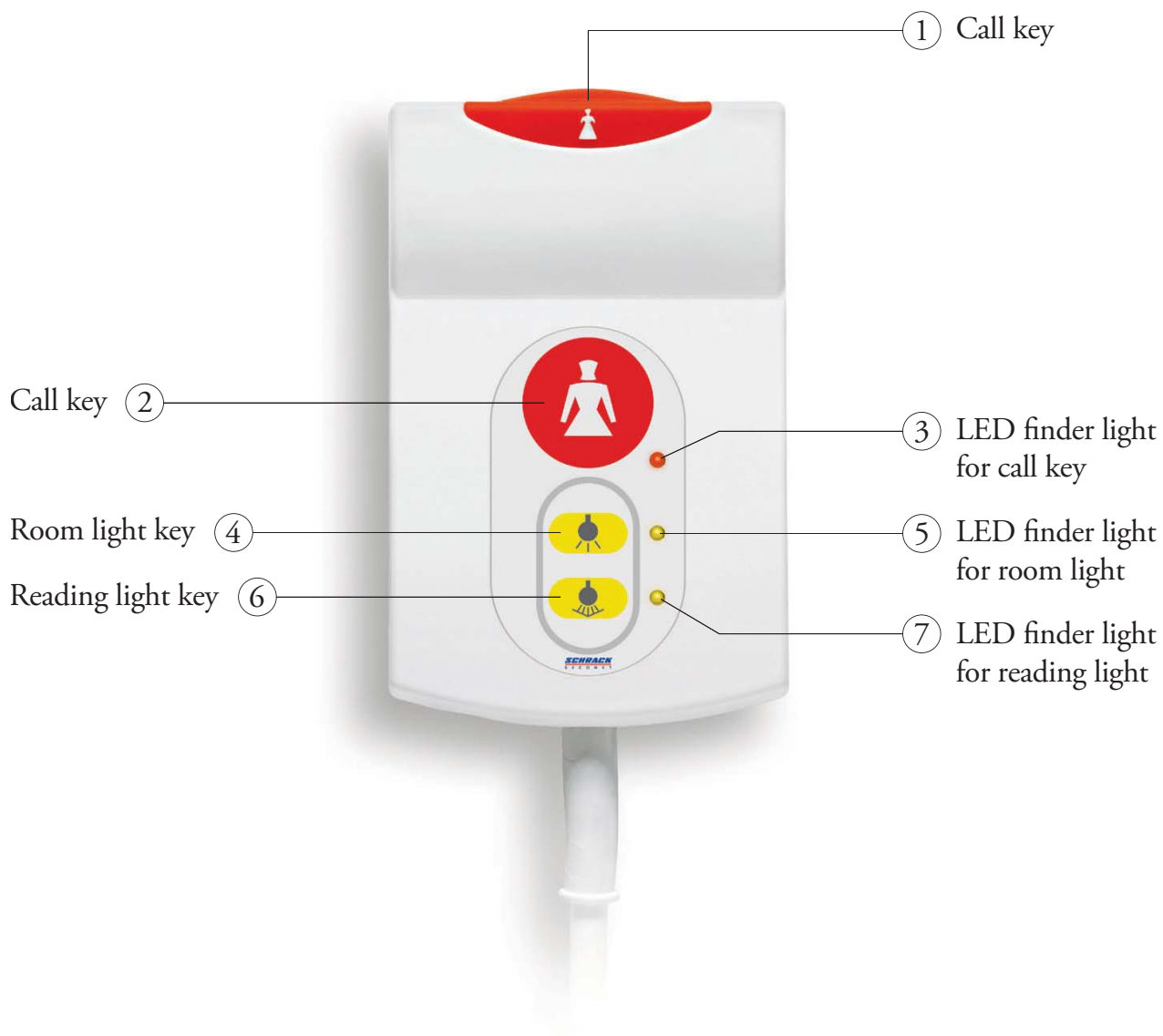

CAUTION: It must be checked that the device is fully functional each time the device is plugged in and each time a disconnection call is reset. Please note that a disconnection call is automatically triggered when the device is unplugged.



Utmost security and a contemporary level of comfort.

The communications system which you are accessing with this operating device, excels thanks to its reliability and the future-proof technology it uses. The best hospital and care facilities around the globe have placed confidence in it.

There is a name for hospital communications – SCHRACK SECONET.

Nurse call


To call nurse:

- Press call key (1) or (2)  
- LED in call key (1) and LED finder light for call key (3) light up


The nurse will come to you in person.

Turn light ON/OFF

Room light (optional):



- Press room light key (4) briefly 
- Room light is turned on/off

Reading light (optional):

- Press reading light key (6) briefly 
- Reading light is turned on/off

Finder light/reassurance light

The LEDs (1, 3, 5 and 7) are constantly glow, so that the keys are easier to locate even in the dark.

If either the call key (1) or the call key (2) are pressed,   then LEDs (1) and (3) light up.

Cleaning and Disinfection

For the BT-B patient handset, without exception only use disinfectants that do not contain ketones or esters either in part or in full. For the disinfection process disinfection by wiping should be used – and under no circumstances a disinfection by submersion process.



SCHRACK SECONET AG

Eibesbrunnnergasse 18
A-1120 Vienna
Tel. +43 1 81157
office@schrack-seconet.com
www.schrack-seconet.com

EN

Czech Rep. • CZ-149 00 Prague 4 - Újezd., Štítová 283 • Tel. +420 2 74784422
Hungary • HU-1119 Budapest, Fehérvári út 89-95 • Tel. +36 1 4644300
India • IN-122002 Gurgaon, DLF Golf Course Road, Sector-54 • Tel. +91 124 4141501
Poland • PL-02-672 Warsaw, ul. Domaniewska 44a, bud. Platinum V • Tel. +48 22 3300620
Romania • RO-023961 Bucharest, Str. Mântuleasa nr. 15A et. 1, Sector 2 • Tel. +40 372 756316
Russia • RU-129626 Moscow, Ul. Staroalexejevskaja 5 • Tel. +7 495 5105015
Slovakia • SK-83527 Bratislava-Rača, Mudrochova 2 • Tel. +421 2 44635595
Sweden • SE-145 84 Norsborg, Botvid Business Center • Tel. +46 8 6801860
Turkey • TR-34722 Kadıköy-İstanbul, Kasap İsmail Sk. 5/5 • Tel. +90 216 3455199

HEALTH CARE

SCHRACK
S E C O N E T